

SWITCHAPP

QuickStart Guide



1


Get your supplies!

1. Laptop
2. Cables/Wires
3. Microphone
4. Mic Box




2

Double-click the icon! to start SwitchApp

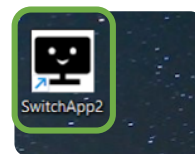

3

Make a new user or Log In

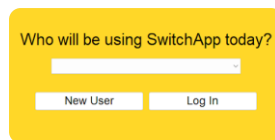

4

Set up my Switch



Record Clips

Pick your 1st sound

This should be a sound that's easy for you to repeat.



Setup your mic box

Check that the audio is working.

Click

and adjust the volume!

Record sound clips!

Click and follow the prompts to record short sound clips.

Your helper should press the space bar each time you say the word!

5

Set up my Switch



Sort Clips

Pick the new session

Select the set of new audio clips to sort them. Play the first clip.




Sort your sounds

Sort the clip by clicking the matching label. There may be clips of "Other Sounds".



Don't forget to save!

Repeat until all the clips are sorted. When you're done, click:

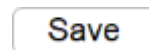

6


Set up my Switch



Review Clips


Pick a sound to check

Double-check that all the sounds you labeled are sorted correctly.

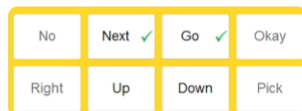

Review your sounds

Play each sound clip and make sure that it matches the label.

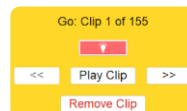

Don't forget to save!

Repeat until all the clips are reviewed. When you're done, click:




7

Test my Switch




Pick a sound to test
Beside "1", press select to choose the sound you prepared.

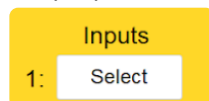


Create a new model

Check the box beside your sound and "Create New Model".

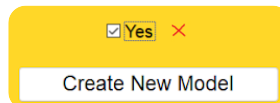

Test the sound!

Test your microphone like in Step 4. Follow the prompts to see how well SwitchApp detects the word!

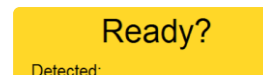

Your personalized switch is now ready to be used with the interface of your choice!

SWITCHAPP

QuickStart Guide

1


Get your supplies!

1. Laptop
2. Cables/Wires
3. Webcam
4. Lighting



2

Double-click the icon! to start SwitchApp

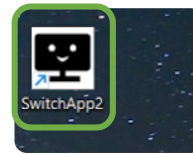

3

Make a new user or Log In

Who will be using SwitchApp today?

4

Set up my Switch



Record Clips

Record your face

Select "No Movement" and press to set up.

Stay still while the app records your face.

Pick your 1st action

This movement should be something easy for you to repeat.



Record movement clips!

Click and follow the prompts to record short video clips.

Your helper should press the space bar each time you do the movement!

5


Set up my Switch



Sort Clips

Pick the new session

Select the set of new video clips to sort them. Play the first clip.

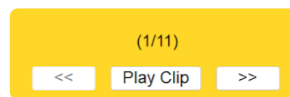

Sort your actions

Sort the clip by clicking the matching label. There may be clips of "Other Movements".



Don't forget to save!

Repeat until all the clips are sorted. When you're done, click:

6


Set up my Switch



Review Clips


Pick an action to check

Double-check that all the movements you labeled are sorted correctly.

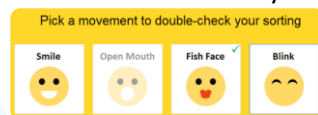

Review your actions

Play each video clip and make sure that it matches the label.

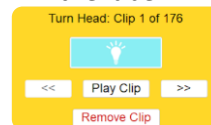

Don't forget to save!

Repeat until all the clips are reviewed. When you're done, click:


7

Test my Switch




Pick an action to test

Beside "1", press select to choose the movement you prepared.

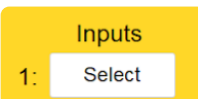

Create a new model

Check the box beside your movement and "Create New Model".

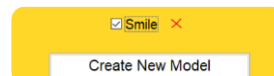

Test the movement!

Set up your camera like in Step 4. Follow the prompts to see how well SwitchApp detects the movement!

Detected:

Your personalized switch is now ready to be used with the interface of your choice!